Premier RC-Prep used with sodium hypochlorite solution gives you these advantages:

- Facilitates cleansing and shaping of the Root Canal. The EDTA helps remove calcium salts from the calcifications and from the canal surface, permitting the reamers and files to more rapidly cleanse and shape the canal.
- Pulp Tissue - Vital and/or necrotic tissue is more easily removed from the canal when irrigated with sodium hypochlorite solution, the oxygen bubbles released from the urea peroxide float out the pulp tissue, shavings and other debris.
- Helps brighten the tooth if discolored in addition to its ability of digesting pulp tissue, sodium hypochlorite solution helps bleach the tooth. This process is enhanced by the oxygen liberated from the urea peroxide.
- Enhances the penetration of medicaments throughout the root canal system by its ability to more completely clean the canal and open the dentinal tubules.

RC-PREP contains Ethylenediamine Tetraacetic Acid. The chelating ability of this product will help remove the calcium salts from pulp stones and from the walls of the root canal. This action aids in the cleansing and shaping of the canal. **RC-PREP** also contains urea-peroxide in a special water soluble vehicle that lubricates the canal permitting instruments to more easily enter even fine canals and reduce the incidence of breakage. **RC-PREP** should be reacted with sodium hypochlorite solution which reacts with the urea-peroxide. The fine bubbling produced by the liberation of oxygen floats out pulpal debris and dentin shavings. The action frees the EDTA to more readily chelate the calcium salts. The effervescent activity helps prevent packing debris into the apical area. It also helps to unclog the dentinal tubules encouraging better penetration of root canal medicaments.

RC-PREP IS DISPENSED IN:

- Jars of: 18g and 227g
- One box of: 2 prefilled syringes - 9g each
- Dispensing Pump 18g
- MicroDose pipettes 0.2mL Qty: 100
- 3mL syringes Qty: 5

Note with Syringe use:

- Disposable tips are designed for single patient use only.
- Use a protective plastic sheath barrier on the syringe in order to mitigate potential cross-contamination between patients.

Storage and Use

- Store in a cool place. (8°C/46°F -15°C/59°F). Avoid excess heat.
- Use only as directed, per indications for use. Tightly recap after use.
- Disinfect syringe using an intermediate level disinfection process (liquid contact) as recommended by the Centers for Disease Control. Guidelines for Infection Control in Dental Health-Care Settings – MMWR, December 19, 2003;52(RR-17), Centers for Disease Control and Prevention.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

Warranty and Limitation of Liability

Premier Dental Products Company ("Company") warrants that the Product will be free from defects in material and manufacture. Company makes no other warranties. In particular, and without limitation, Company makes no warranty, implied or otherwise, as to the Product's merchantability or fitness for any particular purpose.

For any patient on whom use of the Product is contemplated, user is solely responsible for determining whether the Product is suitable for such use according to the information set forth in the enclosed directions for use.

If the Product is defective within the warranty period, the sole remedy available to user, and the sole obligation of Company, shall be the Product's repair or replacement.

Unless otherwise required by law, Company will not be liable for any loss or damage arising from this Product or its use, whether such loss or damage is direct, indirect, special, incidental or consequential, and regardless of the theory of loss or damage asserted (which theories include, without limitation, warranty, contract, negligence and strict liability). Unless otherwise required by law, no party other than the Company is authorized to provide to any other party any information concerning the Product which deviates from the information set forth in the enclosed directions for use.

Active Ingredients:

Ethylenediamine Tetraacetic Acid (EDTA) and Urea Peroxide in a water soluble glycol base.

DIRECTIONS FOR USE (Please follow carefully)

The tooth to be treated should be isolated with a rubber dam. The pulp chamber is opened in the usual manner with every effort made to provide adequate access to the root canal or canals. The chamber contents are removed with sterile burs and excavators and then flushed with sodium hypochlorite solution. The pulp chamber is then filled with RC-Prep. The sodium hypochlorite solution will react with the peroxide and the bubbling will help open the orifices of the canal. A fine instrument, reamer or file, is then introduced into the canal and using a slight pumping or mixing motion will enhance the bubbling and float out debris that is aspirated away. The first instrument should not be placed more than half the way down the canal. This action is then followed by gentle irrigation with sodium hypochlorite solution. The procedure is repeated adding RC-Prep to the pulp chamber. This time, the instrument is carried to approximately 3/4 the distance of the canal. The mild pumping and rotation of the instrument will encourage more debris to be floated from the canal. When a vital pulp is present, it is not unusual to see the entire pulp float to the surface or be removed with the file. After gentle irrigation, the instrument is again inserted to approximately one mm. short of the radiographic apex and an x-ray film taken to confirm the measurement. This measurement should be recorded and/or corrections made. Using this technique reduces the likelihood of inoculating the periapical tissues with the inflamed or infected contents of the canal. It helps reduce post-operative discomfort and helps prevent flare ups. In teeth with two or more roots, the effervescent activity will tend to cleanse the pulp chamber and open the orifices of the canals, making it easier to find the entrance and easier to insert the root canal instruments. Never proceed to the next size instrument until the previous instrument passes freely to the desired position in the canal. The RC-Prep is used in the chamber and canals only with the first two or three sizes of instruments. The canals should be further prepared using sodium hypochlorite solution only with the next one or two instrument sizes. There is usually enough residual activity of the RC-Prep to lift out the remaining pulp tissue and debris. Sodium hypochlorite solution should be used in the canals and repeatedly instrumented into the canals until all evidence of bubbling has ceased. The canals can be dried with absorbent points and the medicaments of choice sealed into the canals and chamber of the tooth. It is not suggested the RC-Prep be sealed into the pulp chamber in the hope of opening calcified canals. Residual organic tissues might react with the peroxide and cause pressure and discomfort. Always irrigate the canals and chamber with sodium hypochlorite solution until all evidence of bubbling has ceased.

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